## HOISTING & STOWING CAROLIFT 100











## **HOISTING & STOWING CAROLIFT 100**





The Carolift 100 hoist was designed to be simple to use yet powerful enough to lift any wheelchair, scooter or other mobility device weighing up to 100 kg into the boot of your car. You control the hoist via an intuitive four-button hand control, and it does its job in about a minute.

Via the hand control, the Carolift 100 hoist does all the heavy work for you. It lifts and rotates the cargo into the boot of your car. All you need to do is lend a slight guiding hand. The Carolift 100 hoist does its job very quickly. Loading or unloading your mobility device can be done in around a minute. All movement is at a smooth and constant pace, you'll be in total control all the time.



The clever design of the Carolift 100 shines through in its ease of use. Open your boot, deploy the lift over your mobility device and lower the lift arm. Once within reach, you connect the clever lifting straps. Use as much slack as you need to fit the buckle on the mobility device. Pulling on the loop will tighten both sides simultaneously. Don't worry if the strap is longer on one side. The Carolift 100 will self-balance the load as soon as it leaves the ground.

## Features



Lift up to 100 kg



Powered lifting and manual swivel



Easy installation thanks to smart mounting base



Operate via control on the hoist or smartphone app



Vehicle maintains full seating capacity



Suitable for most vehicle makes and models.

## Specifications



Lift Capacity: 80 kg - 100 kg



Weight: 22.8 kg (Including mounting bracket)

Voltage: 12V

Swivel Degree: 360°



